


# QUICK GUIDE

---

## EarPenguin Optical Otoscope

EP-001-H

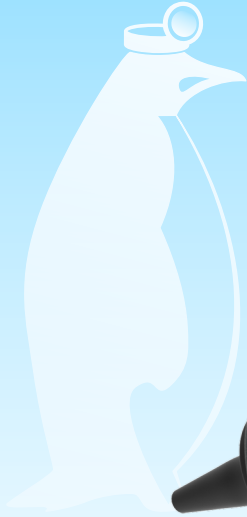


# SIBBO MEDICAL VICES



# WIRELESS CHARGING

---

EarPenguin Optical Otoscope is charged wirelessly. To charge, place the EarPenguin tummy down on a Qi standard charger.

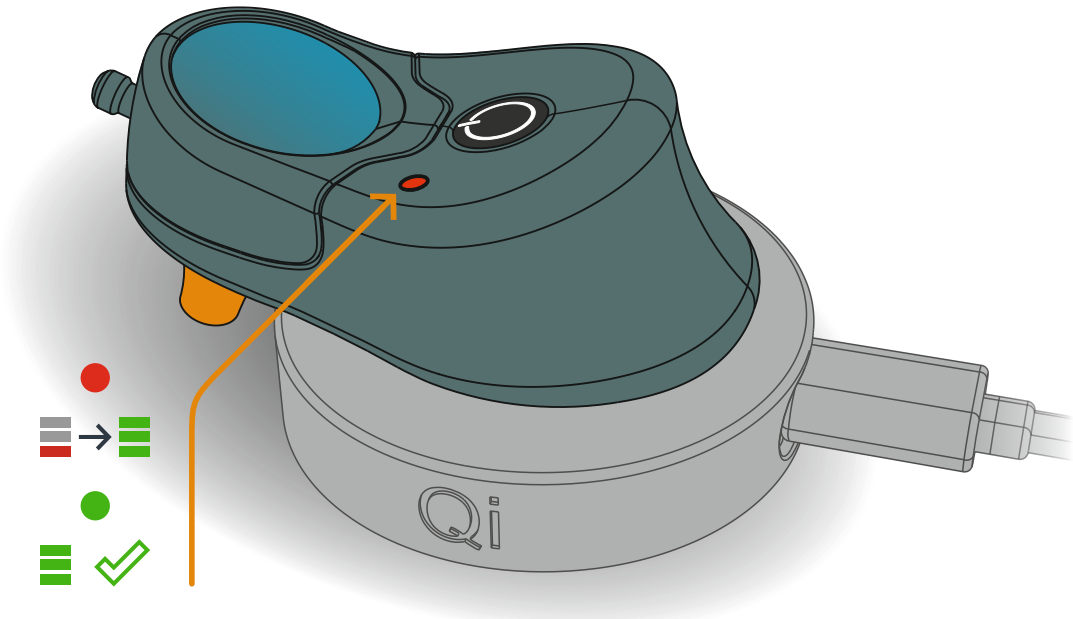


When charging, LED indicator will emit red light.  
Once charging is done, the light will turn green

Any Qi standard charger will do, but we suggest using a 5W -10W charger to reduce heating. Charging should take between 2-4h depending on your charger of choice.

# HOW TO USE

---

Inflation bulb connection.

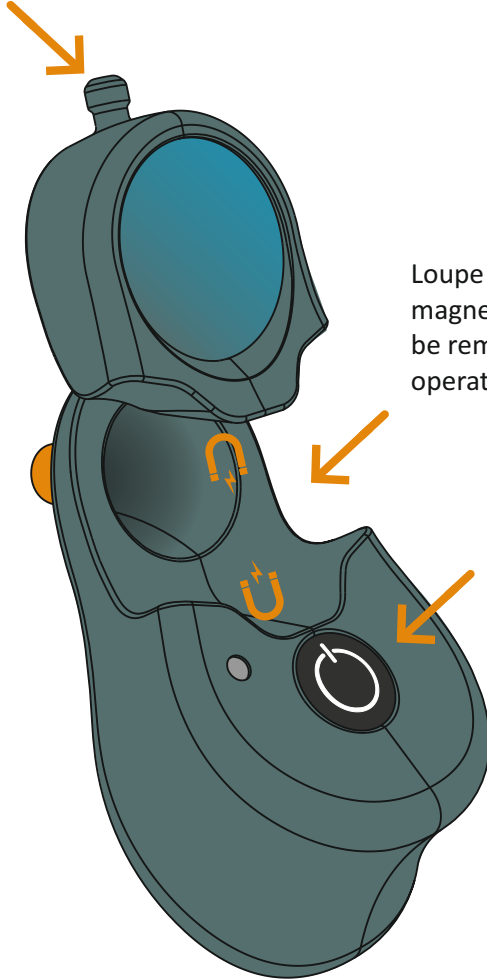

Loupe is held in place magnetically and can be removed for operational access.

On / Off

**Clean the EarPenguin devices by wiping them with a clean cloth and 70% -isopropanol. Use a damp cloth, do not soak.**

# WARNINGS / SPECS

---

## INTENDED USE

The EarPenguin otoscope system consists of the EarPenguin optical otoscope and the EarPenguin digital otoscope to be used with the Sibbo MD monitor. The EarPenguin otoscopes can be used together or as separate stand-alone devices. The EarPenguin otoscopes are intended to be used by clinicians and medically qualified personnel to visualize and capture recordings (digital otoscope only) of the external ear, ear canal and tympanic membrane on pediatric and adult patients.

**Report any serious incidents related to the device immediately to the manufacturer and your local Competent Authority.**



Do not replace the batteries within the device.  
Incorrect replacement may cause a fire and serious injury.



No modification of this equipment is allowed.



Only use batteries supplied with the monitor.



Only charge the monitor batteries with the provided charger.

## Technical specifications

Energy source	LiPo battery	3.7V	500mAh
Charging	Qi standard	5-15W	
Light source	8 LEDs	3000K	
Weight		70g	

## Sibbo Medical Devices Oy

Hitsaajankatu 22

00810, Helsinki

Finland

[info@sibbomedicaldevices.com](mailto:info@sibbomedicaldevices.com)

